## 2-Level Kontrol Kit

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## 2-Level Control Kit

### Intended Use

The CombiScreen® Control PN is intended for use in the clinical laboratory to validate the performance of the CombiScreen® urine test strips and the CombiScan® / Urilyzer® / CombiScreen® urinalysis systems during daily routine testing. The observed values can be compared to the target ranges given in the table below.

### Product Description

The CombiScreen® Control PN is supplied as a 2 level test kit (15 mL level N, 15 mL level P). Level N solution corresponds to a normal urine sample. Level P solution will show a positive result for bilirubin, blood, glucose, ketones, leucocytes, nitrite, protein and urobilinogen. The kit is ready-to-use, there is no dilution or any other operation necessary. The kit is prepared from human urine, compounds that generate the desired reaction when tested with the products mentioned above are added to reach the target level. There are preservatives added to ensure stability and to avoid microbial growth.

### Warnings / Precaution

POTENTIAL BIOHAZARDOUS MATERIAL.

Contains human urine and human chorionic gonadotropin (hCG) from pregnancy urine. The human hCG source material used in the manufacture of this product has been tested and found non-reactive for Hepatitis B Surface Antigen, Hepatitis C and HIV 1&2 antibody when tested by FDA (USA) accepted methods. No known test method can assure that a product derived from human material does not contain Hepatitis or HIV virus. It is recommended that such samples be handled as potentially infectious according to the Centers of Disease Controls (CDC, USA) biosafety Level 2 recommendations.

Warning! H317: May cause an allergic skin reaction. P261: Avoid breathing vapors, mist, or spray. P272: Contaminated work clothing should not be allowed out of the workplace. P280: Wear protective gloves, protective clothing, and eye protection. P302+P352: If on skin: Wash with plenty of water. P333+P313: If skin irritation or rash occurs: Get medical advice/attention. P362: Take off contaminated clothing and wash it before reuse. P501: Dispose of contents/container in accordance with local, regional, national, and international regulations.

The material safety data sheet contains further safety-related information. It is available for download from our homepage <http://www.analyticon-diagnostics.com>.

### Storage and Stability

Store CombiScreen® Control PN at 2-8 °C in a dark place. Do not freeze!  
When stored unopened at 2-8°C, the product is stable until the expiration date printed on the label. After initial use, each tube of control is stable for 3 months or 20 dipstick immersions, whichever occurs first.  
If the controls are turbid or if any evidence of microbial growth or contamination is present, the controls should be discarded. This should be done according to the local guidelines in the same manner as for other biological specimens.

### Procedure

Remove the CombiScreen® Control PN from the refrigerator and let it come up to room temperature (20 - 25 °C) at a dark place. This should take minimum 15 minutes, depending on the remaining volume. Mix gently to assure homogeneity, but avoid foaming.

Remove the cap from the test solution tube, and use the solution as if it were a patient sample.

Open test strip tube and take out test strips, close tube immediately, do not store strips outside the tube, and avoid influence of direct sunlight.

Immerse test strip approx. 2 seconds into test solution, so that all test pads are covered with test solution.

Remove excess of test solution by wiping the edge of the strip on the test solution container.

To prevent interactions from adjacent test pads, hold the test strip in a horizontal position during incubation, or place it in the instrument as described in the manual.

Compare the reagent areas on the test strip with the colour chart after 60 seconds or check the printout of the instrument.

Use diffuse daylight for visual evaluation. Direct sunlight or light of light bulbs is not recommended.

The Urobilinogen test pad may show a more beige-orange colour compared to the label. Avoid repeated dipping in the same container with test solution level N without homogenizing it, because this may cause false positives in nitrite and blood.

Close the controls and return them to the 2 - 8 °C storage.

### Notes

Check also the instructions of the CombiScreen® package insert and the CombiScan® / Urilyzer® instrument manuals.

Avoid swallowing and contact with mucous membranes, skin or eyes!  
Don't drink! Keep away from children!

For in-vitro-diagnostic use only! For professional use only! Not for self-testing!

### Expected readings

The target ranges have been determined with different LOTs of CombiScreen® test strips and different CombiScan® instruments under different conditions. They can be taken from the table below. Each laboratory should establish its own precision parameters.

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